

Selecting A Level

App2 Talk will prompt the user to select a level at initial set up. The level can be changed at any time. **The user must touch Save after selecting a level.**

Beginner: Communicate basic needs and wants with a single word at a time. Ideal for those who have never used an app like App2Talk before and communicate minimally.

Intermediate: Introduces descriptive and question words to better give and get information. Communicate with a single word or form a simple sentence.

Advanced: Use the full App2Talk vocabulary to create sentences, including words for social interactions. Sub-categories help further organize words.

Select a Category/Sub-Category

1. When the user opens App2Talk the “Fast Talk” page will be the first screen the user will see.
2. To go to the “Categories” page, touch the category bar (Fast Talk) at the top middle of the screen.
3. Scroll left or right using a finger swipe or the Next Page and Previous Page buttons to choose a category.
4. Touch the desired category. The user will hear the name of the category and the category will open to a sub-category (if available) or a sticker page.
 - A long press/touch will result in the user hearing the name of the category only.
5. Scroll left or right using a finger swipe to choose a sub-category. **Sub-categories are not available for all categories.**
 - Touch the desired sub-category. The user will hear the name of the sub-category and the sticker page will open.
6. Stickers will be displayed.
7. To return to the “Categories” page, touch the category bar at the top middle of the screen.

Select a Sticker

1. Once the user opens a category/sub-category the stickers will be displayed.
2. Scroll left or right using a finger swipe or the Next Page and Previous Page buttons to choose a sticker.
3. The selected sticker will be displayed in the message bar.
4. To return to the categories page touch the category bar at the top middle of the screen.

Message Bar

Tile Icon:  The tile icon displays one sticker at a time. Selecting a new sticker replaces the existing sticker in the message bar.

Pencil Icon:  The pencil icon displays multiple stickers. The user also has the ability to start, stop, delete, and/or replay a message.

Change the # of stickers displayed per page

1. Select any category/sub-category.
2. Touch the 3 lined icon  in the top right hand corner.
3. The screen will slide to the left revealing a menu on the right hand side.
4. Select the number of stickers that will be displayed on each page.
5. Touch the 3 lined icon again to return to sticker page.

Edit Categories

1. From the “Categories” page, select a category/sub-category.
2. Touch the 3 lined icon  in the top right hand corner.
3. A menu will open on the right hand side.
4. Touch Edit Categories.
5. A list of all categories will be displayed with an on/off icon to the right.
6. Swipe left or right to view all categories.
7. To turn off a category, tap the on/off icon until the purple color disappears. This category will no longer be displayed on the “Categories” page until turned on again.
8. To turn the category back on, tap the on/off icon until the purple color re-appears.

Edit Existing Stickers

1. Follow steps 1-5 under Edit Categories above.
2. Touch the name of the category/sub-category to be edited.
3. All stickers will be displayed in that category/sub-category. Scroll up and down to see all stickers.
4. Touch Edit at the top right corner of the page.
5. Touch an existing sticker to edit.
6. Touch image to replace. Camera will allow the user to take a customized picture. Image Library will pull pictures from the device.
7. Touch label bar to re-name.
8. Use recording feature to record a personalized voice. If a personalized voice is not recorded, the app will automatically use Siri.
9. Touch Save.
10. Touch Back to return to the “Pick a Sticker” page.

11. Touch Back to return to the “Edit Categories” page.

** There is not a home button. The user must use the Back button to return to previously selected pages.

Add New Stickers

1. Follow steps 1-5 under Edit Categories above.
2. Touch the category/sub-category name to be edited.
3. All stickers will be displayed in that category/sub-category. Scroll up and down to see all stickers.
4. Touch Edit at the top right corner of the page.
5. Scroll to the bottom of the stickers until you see Add New with a camera icon.
6. Touch Add New.
7. Touch camera icon to add an image.
8. Enter name under the image.
9. Use recording feature to record a personalized voice. If a personalized voice is not recorded, the app will automatically use Siri.
10. Touch Save.
11. Touch Back to return to the “Pick a Sticker” page.
12. Touch Back to return to the “Edit Categories” page.

** There is not a home button. The user must use the Back button to return to previously selected pages.

Add Existing Stickers

1. Follow steps 1-5 under Edit Categories above.
2. Touch the category/sub-category name to be edited.
3. All stickers will be displayed in that category/sub-category. Scroll up and down to see all stickers.
4. Touch Edit at the top right corner of the page.
5. Scroll to the bottom of the stickers until you see Add Existing with a tile icon.
6. Touch Add Existing.
7. Touch the empty circle in the top right corner to add the sticker to a page. The circle will turn green with a white plus sign when selected.
8. Touch Save.
9. Touch Back to return to the “Pick a Sticker” page.
10. Touch Back to return to the “Edit Categories” page.
11. The added sticker will now appear on the category page.

** There is not a home button. The user must use the Back button to return to previously selected pages.

Arrange Stickers – In Edit Mode

- Hold sticker then drag and drop in desired location.
- Touch check mark in top right corner to hide a sticker. When the sticker is unchecked, the image will not appear on the page. Touch the empty circle in the top right corner to re-check the sticker when the user wants it to re-appear on the page.
- Scroll to the bottom of the stickers until you see an empty sticker. Drag and drop the empty sticker in any desired location.

**Stickers cannot be deleted, only moved and/or hidden.

Add a User

1. From the “Categories” page, select a category/sub-category.
2. Touch the 3 lined icon  in the top right hand corner.
3. A menu will open on the right hand side.
4. Touch the user name/image.
5. Touch Add User icon.
6. Touch camera to add image.
7. Enter username under image.
8. Choose level (Beginner, Intermediate, and Advanced).
9. A check mark will appear in the top left hand corner of the level selected.
10. Touch Save.
11. Touch Back to return to the “Select A User” page. The selected user will be highlighted.
12. Touch Back to return to the app.

** A user profile can be added or edited at any time, but it cannot be deleted.

Switch Users

1. From the “Categories” page, select a category/sub-category.
2. Touch the 3 lined icon  in the top right hand corner.
3. A menu will open on the right hand side.
4. Touch the user name/image. “Select A User” page will open.
5. Touch the user you would like to switch to.
6. The selected user will be highlighted.
7. Touch Back to return to the app.

Edit a User

1. From the “Categories” page, select a category/sub-category.
2. Touch the 3 lined icon  in the top right hand corner.
3. A menu will open on the right hand side.
4. Touch the user name/image. “Select A User” page will open.
5. Touch Edit to the right hand side of the user you would like to edit.
6. Edit image, name, animate sticker selection, and/or level.
7. Touch Save.
8. Touch Back to return to the “Select A User” Page.
9. Touch Back to return to the app.

Export Data

1. From the “Categories” page, select a category/sub-category.
2. Touch the 3 lined icon  in the top right hand corner.
3. A menu will open on the right hand side.
4. Touch Export Usage Data.

App2Talk Tutorial Instructions

5. Select which user's data you would like to export.
6. Enter an email address.
7. Touch Send.